



X-RAY, CT, ULTRASOUND, MRI, NUCLEAR MEDICINE, BMD, OPG, ECHOCARDIOGRAPHY,
HOLTER, AMBULATORY BP, EXERCISE STRESS TESTS, CARDIOLOGY CONSULTATIONS

North Queensland X-Ray Services now offers IntelRad (IntelConnect and IntelViewer)
for you to view reports and images

Apply for an account at

<https://imaging.nqxray.com.au>

(save as a desktop shortcut on your workstation)

View reports and images using IntelConnect (web version of IntelViewer)

- Open Chrome or Browser of Choice
- Type in <https://imaging.nqxray.com.au>
- Log in with your account details for NQXRAY

How to Break-glass with IntelConnect

For those users who view North Queensland X-Ray Services' IntelRad PACS, should you wish to view images and reports referred by another practitioner, you can break glass .

- Go to <https://imaging.nqxray.com.au> and log in with your account details
 1. Hit the break-glass(padlock) button at the top of the window next to where you enter patient details
 2. Then follow the instructions
 3. You need to type in the patient details correctly, ie Smith,John for name and 19801203 for DOB (YYYYMMDD)

Having issues? Email intelepacs@nqxray.com.au



X-RAY, CT, ULTRASOUND, MRI, NUCLEAR MEDICINE, BMD, OPG, ECHOCARDIOGRAPHY,
HOLTER, AMBULATORY BP, EXERCISE STRESS TESTS, CARDIOLOGY CONSULTATIONS

How to install IntelViewer and add a bookmark for North Queensland X-Ray Services

To install IntelViewer , go to
<https://imaging.nqxray.com.au/IntelViewer>

1. Open IntelViewer
2. *Edit Bookmarks*
3. *Add*
4. type in URL <https://imaging.nqxray.com.au>
5. click on *Protect Bookmark*
6. Once you have signed in (this may upgrade IntelViewer to the latest version and you may require your department IT admin permission)
7. go to *Utilities*, select *Automatic Login* and re-type your Password – you no longer require to enter Password each time you use NQXRAY bookmark in IntelViewer

How to Break-glass in IntelViewer

For those users who view North Queensland X-Ray Services' IntelRad PACS, should you wish to view images and reports referred by another practitioner, you can break glass

1. Go to the Search tab in IntelViewer
2. Left-mouse button click on "Break-glass Access"
3. Agree to confidentiality statement
4. Left-mouse button click on " Patient name & date of birth"
5. Then Search for patient using Surname, Firstname and YYYYMMDD for date of birth eg for May 09,2000 you would type 20000509

Having issues? Email intelepacs@nqxray.com.au